



Quick-Start Guide

LTE Blood Pressure Monitor

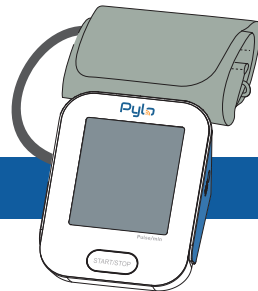

PYLO 802-LTE


1

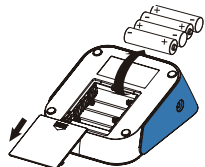

Insert the 4 included AA batteries into the device.

Once inserted, the device will initiate an initial secure pairing with the mobile network.

2

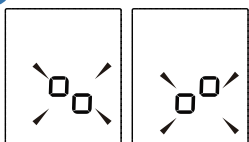


Pairing in progress

While pairing, the symbols $\square \cdot$ and $\cdot \square$ will be shown.

This process normally takes around 10 seconds, but may take up to a minute if signal is weak.

3

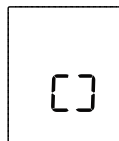


Secure Pairing Successful

When pairing is completed successfully, the device will display the $\square \cdot$ symbol and the screen will turn off.

The device is now ready to use!


4

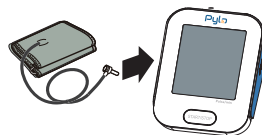

Plug the cuff tube into the device.

Be sure to use the port labeled 'Cuff Port' and not the port for the optional power adapter.

5

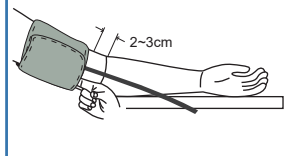

Wrap the cuff around your arm and position it before tightening.

Be sure to read and follow the "How to Take Your BP" guide that was included with your monitor.


6



Press the "START/STOP" button to turn on the monitor. The cuff will inflate, take a reading and then deflate.


Remain still until the cuff deflates completely.


7

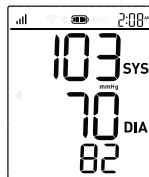

Your results will be displayed and saved to the device prior to transmission starting.


8

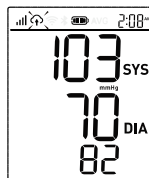


During data transmission the  symbol will flash.

This process normally takes around 10 seconds but may take longer when signal weak.

9

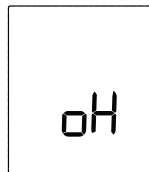

Transmission Successful

If the transmission is a success,  is shown.

If connection fails, E5 or E6 is displayed. In this case, the measurement is saved and will be sent on the next successful connection.



For more information, see the full instruction manual.

Pylo